

InviroSNAP![®] Safety Syringe

Usage Guidelines

PREPARATION FOR DOSING

- ▶ **Peel package as indicated**, to allow proper removal of SNAP![®] Safety Syringe.
- ▶ Remove the needle cap. Draw the medication and expel the air bubbles using standard clinical technique.

DOSE ADMINISTRATION

- ▶ Prepare injection site and administer dose using standard technique.
- ▶ **Depress plunger fully.**
- ▶ **Needle cannot be retracted if plunger is not fully depressed.**

SNAP![®] INSTRUCTIONS FOR DISABLING



▶ **PUSH**

Depress plunger fully to engage safety mechanism to enable retraction of needle.



▶ **PULL**

Pull back on plunger until it stops; observe withdrawal of needle into the barrel, and offset of needle in barrel.



▶ **SNAP!**

Snap off plunger to permanently disable syringe.

SYRINGE DISPOSAL

- ▶ Discard the used Inviro SNAP![®] Safety Syringe per protocol in a Sharps container.

This device is sterile unless the package has been damaged or opened. This is a single use item only.

InviroSNAP!® Seringue de sécurité

Mode d'emploi

ÉTAPES DE PRÉPARATION

- ▶ Ouvrir l'emballage comme indiqué, pour retirer de celui-ci la seringue de sécurité SNAP!® de façon correcte.
- ▶ Enlever le capuchon de l'aiguille. Aspirer le médicament et expulser les bulles d'air conformément à la pratique clinique usuelle.

ADMINISTRATION D'UNE DOSE

- ▶ Préparer le site d'injection et administrer la dose suivant la technique usuelle.
- ▶ **Enfoncer complètement le piston.**
- ▶ **Il sera impossible de rétracter l'aiguille si le piston n'est pas complètement enfoncé.**

SNAP!® MISE HORS D'USAGE DE L'AIGUILLE



▶ ENFONCER

Enfoncer complètement le piston afin d'activer le mécanisme de sécurité permettant la rétraction de l'aiguille.



▶ TIRER

Tirer sur le piston jusqu'à ce qu'il se bloque tout en suivant la rétraction et le décalage de l'aiguille dans le corps de la seringue.



▶ CASSER!

Casser le piston et la seringue sera définitivement inutilisable.

MISE AU REBUT

- ▶ Jeter la seringue de sécurité usagée Inviro SNAP!® dans un contenant pour objets tranchants ou pointus, conformément au protocole en vigueur.

Cet objet est stérile à moins que l'emballage n'ait été endommagé ou ouvert. Cet article est pour usage unique seulement.